


Opera® Solo Collection Handset Guide

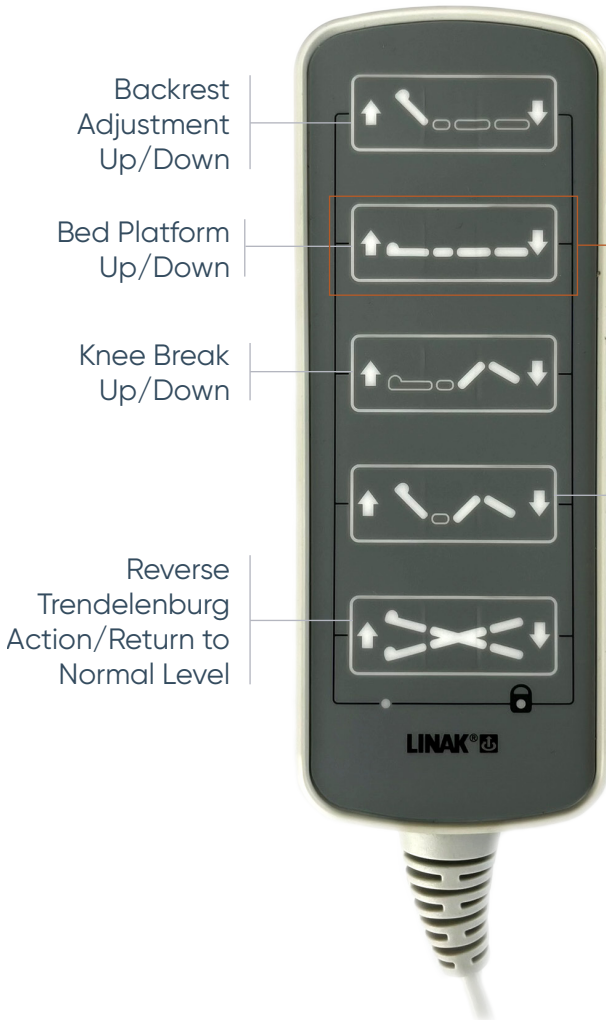

Backrest
Adjustment
Up/Down

Bed Platform
Up/Down

Knee Break
Up/Down

Reverse
Trendelenburg
Action/Return to
Normal Level

ATTENTION

To lower the bed to floor level, press the height down button until it stops at 32cm from floor level. Please check the surroundings are clear and press both height up and down buttons simultaneously to go to floor level.

Backrest &
Knee Break
Up/Down

ATTENTION

To avoid damage, the handset should always be hung up (e.g. on the mattress base or side rail) when not in use.

Press only one button at a time, pressing multiple buttons at one time, may overload and damage the system.

Whilst operating the bed, the rear castors of the bed must be braked to ensure stability during bed adjustments. The Opera Solo Bed should only be moved to reposition the bed.



To lock the handset, wave the locking key along the bottom of the handset near the "Linak" logo. When the handset is locked the Lock Indicator light will turn on.

